# Steri-Pak LF Disposable Bowie-Dick Test Pack

REF BD 108

## Directions For Use

### INDICATIONS FOR USE

The Steri-Pak LF Bowie-Dick Test Pack is for daily monitoring of pre-vacuum steam sterilizers operating at 134°C. The BD 108 model is packaged in an easy to open box.

### COLOR CHANGE

The indicator sheet will change from blue to black when exposed to steam sterilization cycle with acceptable vacuum cycles.

### CRITICAL PARAMETERS

Steam sterilization cycle functioning at 134°C for 3.5 – 4 minutes

### INSTRUCTIONS FOR USE

1. Run a shortened cycle (omitting the drying phase) to properly heat sterilizer.
2. Place the disposable Steri-Pak LF on bottom of sterilizer nearest the drain. Test pack box should be facing up (indicator example graphic on bottom).
3. Run a Bowie-Dick test cycle as specified by sterilizer manufacturer.
4. Run test for 3.5 minutes at 134°C. Sterilization cycle time must never exceed 4 minutes.
5. A short dry time of 2 to 5 minutes will not affect the test.

### INTERPRETATION

1. Remove test pack from sterilizer and open immediately. Pull easy open zip tab across box front to open.
2. Remove pack from box and pull indicator sheet from middle. Dispose of remaining pack materials.
3. If sterilizer is functioning normally, indicator graphic will show a uniform color change (black) throughout the entire pattern.
4. When a significant amount of residual air is present, indicator shows a bluish area in center.

### SAFETY PRECAUTIONS

*CAUTION: the test pack will be hot and should be opened carefully to avoid thermal injury.*

### STORAGE

- Store at normal room temperature 50° - 100°F (10° - 38°C) and 10-70% R.H.
- Protect from moisture and excess humidity.
- Store exposed indicator sheet away from light sources to prevent reversion back to original blue color.

### EXPIRY DATE

The expiry date is printed on the product packaging.

### LOT NUMBER

A unique identification code, **LOT**, is printed on each indicator and packaging labels.

### INTERFERING SUBSTANCES OR CONDITIONS

There are NO KNOWN INTERFERING SUBSTANCES OR CONDITIONS that could affect the intended use of the indicator or adversely affect the indicator performance.

### RELEASE OF TOXIC SUBSTANCES

The indicator releases NO KNOWN TOXIC SUBSTANCES in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

### DECLARATION OF CONFORMITY

SteriTec Products Mfg. Co., Inc. declares that the Steri-Pak LF Bowie-Dick Test Pack indicator conforms to all applicable portions of ISO 11140-1 Class 2 indicators for use in specific tests. In addition, SteriTec Products, Inc. complies with the requirements set out by the U.S. Food and Drug Administrations' (FDA) Quality System Regulation for manufacturing and testing procedures.

SteriTec Products Mfg. Co., Inc.  
74 Inverness Drive East  
Englewood, CO 80112  
(303) 660-4201 FAX (303) 660-4213

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