Steri-Pak Kit LF Sub-Trans Bowie-Dick Test Pack

REF BD 109/S-T

Directions For Use

INDICATIONS FOR USE

The Steri-Pak Kit LF Sub/Trans Bowie-Dick Test Pack is designed for daily monitoring of pre-vacuum steam sterilizers operating at 134° C with sub-atmospheric or trans-atmospheric cycles as defined in ISO 11140-4. The pack materials were designed for use in multiple test cycles.

COLOR CHANGE

The indicator sheet will change from blue to black when exposed to steam sterilization cycle with acceptable vacuum cycles.

CRITICAL PARAMETERS

Steam sterilization cycle functioning at 134° C for 3.5 – 4 minutes

INSTRUCTIONS FOR USE

1. Run a shortened cycle (omitting the drying phase) to properly heat sterilizer.
2. Place one Steri-Pak LF Bowie-Dick indicator sheet between two reusable test pads.
3. Record date of test on test record card (numbered circles 1-15).
4. Insert test packet with centered indicator sheet into stainless steel holder (REF BD 105) and close clamps on both ends.
5. Place metal holder with test pack on bottom of sterilizer nearest drain. The holder must lay horizontal.
6. Run a Bowie Dick test cycle as specified by sterilizer manufacturer.
7. Run test for 3.5 minutes at 134° C. Cycle time must never exceed 4 minutes.
8. A short dry time of 2 to 5 minutes may be run. It will not affect test.

INTERPRETATION

1. Remove the test pack from the sterilizer and open holder immediately.
2. Remove indicator sheet from between the pads. Leave pads in holder, unclamped, for use with the next test.
3. If the sterilizer is functioning normally, the indicator graphic will show a uniform color change to black throughout the entire pattern.
4. If a significant amount of residual air is present during testing, the indicator shows a blue area in the center.

SAFETY PRECAUTIONS

The metal test pack holder will be hot and should be opened carefully to avoid thermal injury. Wear insulated gloves to protect hands.

STORAGE

- Store at normal room temperature 50° - 100°F (10° - 38°C) and 10-70% R.H.
- Protect from moisture and excess humidity.
- Store exposed indicator sheet away from light sources to prevent reversion back to original blue color.

EXPIRY DATE

The expiry date is printed on the product packaging.

LOT NUMBER

A unique identification code, LOT, is printed on each indicator and packaging labels.

INTERFERING SUBSTANCES OR CONDITIONS

There are NO KNOWN INTERFERING SUBSTANCES OR CONDITIONS that could affect the intended use of the indicator or adversely affect the indicator performance.

RELEASE OF TOXIC SUBSTANCES

The indicator releases NO KNOWN TOXIC SUBSTANCES in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

DECLARATION OF CONFORMITY

SteriTec Products Mfg. Co., Inc. declares that the Steri-Pak Kit LF Sub/Trans Bowie-Dick Test Pack indicator conforms to all applicable portions of ISO 11140-1 Class 2 indicators for use in specific tests. The Steri-Pak Kit LF Sub/Trans Bowie Dick Test Pack has been tested according to ISO 11140-4 and fully complies with the performance requirements after exposure to dry saturated steam when tested under the conditions specified in ISO 11140-4. This also certifies that the above testing was conducted in a steam sterilizer according to ISO 11140-4 and using Air Injection Apparatus as described in ISO 11140-4.

*SteriTec Products Mfg. Co., Inc.
74 Inverness Drive East
Englewood, CO 80112 USA
(303) 660-4201 FAX: (303) 660-4213*

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Fabbricante: SteriTec Products Mfg. Co., Inc., Englewood, CO 80112 (USA)