



# INTERPRETATION GUIDE

## Green Card—BD 117


**UNEXPOSED**

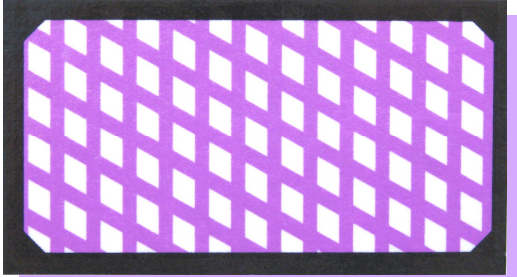


**PASS** No Fault

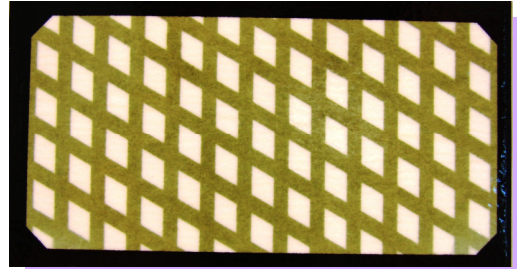


**FAIL** Small Fault

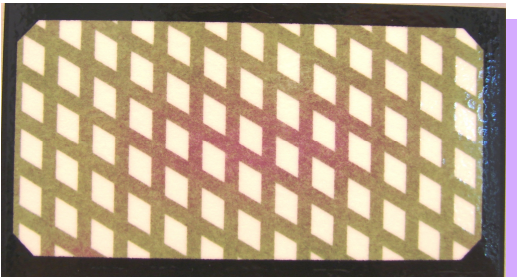


**PASS** Short to medium dry time

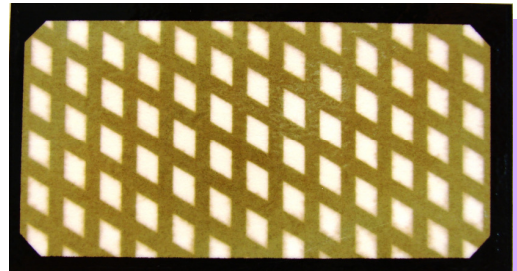


**FAIL** Medium Fault

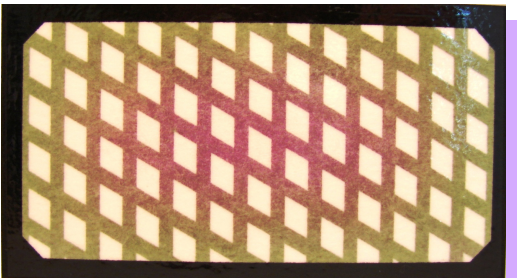


**PASS** Excessive dry time. Shorten dry time to 5 minutes or less for Bowie Dick Test.

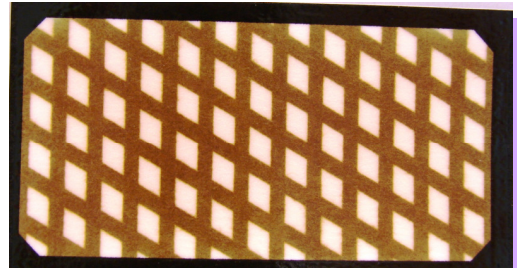

**FAIL** Large Fault

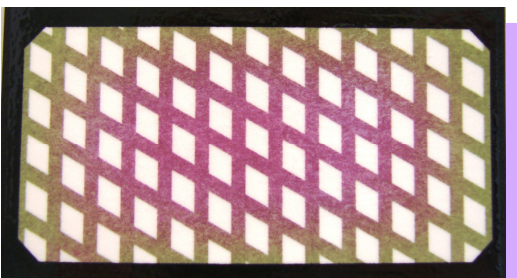

**PASS** Water Spray. Place card in holder up-side down, if steam cannot be dried.

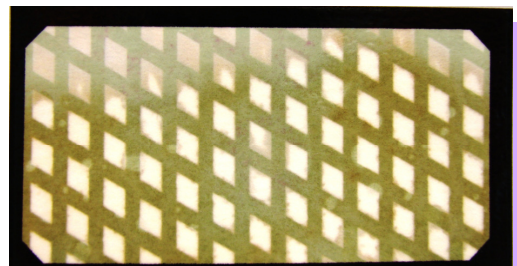

74 Inverness Drive East - Englewood, Colorado 80112

Phone: 303-660-4201 - Fax: 303-660-4213 - E-mail: [info@steritecproducts.com](mailto:info@steritecproducts.com) -

Website: [www.steritecproducts.com](http://www.steritecproducts.com)

MK-074 Rev. A