Dry Heat Indicator Labels

REF CI 113

Directions For Use

INDICATIONS FOR USE

SteriTec Dry Heat Indicator Labels are designed for use in dry heat (hot air) sterilizers. The Dry Heat Indicator Label will alert the user to possible sterilizer malfunction or personnel error. The label provides clear identification of processed and unprocessed items. When used as directed, SteriTec Dry Heat Indicator Labels give visible confirmation that the sterilized item was exposed to dry heat. Under dry heat conditions, the green indicator bar turns from green to dark brown/black.

INSTRUCTIONS FOR USE

1. Clean all items thoroughly in accordance with standard procedures prior to sterilization.
2. Affix one Dry Heat pressure sensitive indicator label to each item to be sterilized.
3. Sterilize package as usual in the dry heat sterilizer.
4. Chemical indicator ink changes color from green to dark brown/black during processing in a dry heat sterilizer.
5. If the indicator bar has not completely changed color from green to dark brown/black, verify that correct sterilizing procedures were followed and report these results to your supervisor.

STORAGE

SteriTec Dry Heat Indicator Labels should be stored at normal room temperatures: 50° - 100° F (10°-38° C). Protect from moisture and excess humidity.

EXPIRY DATE

The expiry date is printed on the product packaging.

LOT NUMBER

The first four digits of the lot number identify the date of manufacture, i.e. a lot number of 0001xx indicates that the product was manufactured in the year 2000 the month of January.

*SteriTec Products Mfg. Co., Inc.
74 Inverness Drive East
Englewood, CO 80112 USA
(303) 660-4201, FAX (303) 660-4213*

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ID&CO S.r.l. - Via Bracciano, 17 - 20098 San Giuliano