## Cross-Checks F

REF CI 118

### Directions For Use

#### INDICATIONS FOR USE

SteriTec Cross-checks F chemical process indicator strips are designed to be used in a LTSF (low temperature steam/formaldehyde) sterilizer. Their purpose is to give visual confirmation of the process of LTSF sterilization conditions at their location within the sterilizer chamber.

#### COLOR CHANGE

During sterilization, the indicator changes from purple to green.

#### INSTRUCTIONS FOR USE

1. Place a Cross-Checks F process indicator strip in each pack, peel pouch, or tray to be steam/formaldehyde sterilized.
2. Process the package as usual in LTSF sterilizer.
3. After processing, visually verify that the indicator spots have changed color from purple to green.

#### SAFETY PRECAUTIONS

- *If there is any doubt about sterility of item, it must be considered NOT sterile.*
- *Do not use any strips whose indicator marks are not purple in color prior to exposure to sterilization processing.*

#### STORAGE

SteriTec Cross-Checks F LTSF process indicator strips should be stored at room temperature 50°- 100°F (10° - 38°C) and normal humidity (10-90% R.H.)

#### EXPIRY DATE

The expiry date is printed on the product packaging.

#### LOT NUMBER

A unique identification code, **LOT**, is printed on each indicator strip and packaging labeling.

#### INTERFERING SUBSTANCES OR CONDITIONS

Do not store near or expose to any formaldehyde or aldehyde based chemicals.

#### RELEASE OF TOXIC SUBSTANCES

The indicator releases NO KNOWN TOXIC SUBSTANCES in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

#### DECLARATION OF CONFORMITY

SteriTec Products Mfg. Co., Inc. declares that the Cross-Checks F indicator conforms to all applicable portions ISO 11140-1

*SteriTec Products Mfg. Co., Inc.  
74 Inverness Drive East  
Englewood, CO 80112 USA  
(303) 660-4201 FAX (303) 660-4213*

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**ID&CO S.r.l.**