STERITEC DRY CHECKS

Dry Heat Indicator Strips

REF: CI 136

Directions For Use

INDICATIONS FOR USE

SteriTec Dry Checks Indicator strips are designed for use in dry heat (hot air) sterilizers. The Dry Check Indicator strips will alert the user to possible sterilizer malfunction or personnel error. The strip provides clear identification of processed and unprocessed items. When used as directed, SteriTec Dry Checks Indicator strips give visible confirmation that the sterilized item was exposed to dry heat. Under dry heat conditions, the light green indicator bar turns to black.

INSTRUCTIONS FOR USE

1. Clean all items thoroughly in accordance with standard procedures prior to sterilization.
2. Place one Dry Check indicator strip in each pack or tray to be sterilized.
3. Sterilize package as usual in the dry heat sterilizer.
4. The chemical indicator ink changes color from green to black during processing in a dry heat sterilizer.
5. If the indicator has not completely changed color from green to black, verify that correct sterilizing procedures were followed and report these results to your supervisor.

STORAGE

Store SteriTec Dry Check Indicator strips at normal room temperatures 50° to 100° F (10°-38° C). Protect from moisture and excess humidity. When stored under these conditions, the Dry Check indicators have a shelf life of three years from the date of manufacture.

Manufactured By:
SteriTec Products Mfg. Co., Inc.
74 Inverness Drive East
Englewood, CO 80112
(303) 660-4201 FAX: (303) 660-4213

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