



## Container Card

REF CI 138

### Directions For Use

#### INDICATIONS FOR USE

SteriTec Container Card chemical indicators are designed for use in steam sterilizers operating at 121° C to 134° (250° F to 273° F). When used as directed, the SteriTec Container Card indicator gives visible confirmation that the container was exposed to steam.

#### COLOR CHANGE

Following sterilization, the indicator turns from white to black.

#### INSTRUCTIONS FOR USE

1. Complete appropriate information on Container Card.
2. Slide a SteriTec Container Card in each container card holder for sterilization. Adjust the card size by folding along the perforated top edge.
3. Sterilize container as usual in a steam sterilizer.
4. Following sterilization cycle, verify that indicator bar has turned from white to black.

#### SAFETY PRECAUTIONS

*Caution: Do not use cards when indicator bar is not white in color prior to exposure to steam sterilization processing.*

#### STORAGE

The SteriTec Container Cards should be stored at room temperature 50°-100° F (10°-38° C) and normal humidity (30-60% R.H.).

#### EXPIRY DATE

The expiry date is printed on the product packaging.

*SteriTec Products Mfg. Co., Inc.  
74 Inverness Drive East  
Englewood, CO 80112  
(303) 660-4201 FAX: (303) 660-4213*