Cross-Checks LS

REF CI 139

Directions For Use

INDICATIONS FOR USE

The SteriTec Cross-Checks LS multi-variable (ISO 11140-1, Class 4) chemical indicator strips are designed for use in pre-vacuum steam sterilizers operating at 121°C (250°F), 132°C (270°F), and 134° C (273°F) for 10, 3.5 or 3 minutes or longer. When used as directed, the SteriTec Cross-Checks LS indicator strips give visible indication that sterilizing conditions were met.

COLOR CHANGE

During steam sterilization, the check mark at the end of the strip changes from Purple to Green.

CRITICAL PARAMETERS (in a standard hospital pre-vacuum steam sterilizer)

Time at 134°C	Time at 132°C	Time at 121°C
3 minutes or longer	3.5 minutes or longer	10 minutes or longer

STATED VALUES (As determined in a steam sterilization resistometer.)

Time at 134°C	Time at 132°C	Time at 121°C
5 minutes	5.5 minutes	15 minutes

INSTRUCTIONS FOR USE

1. Place a SteriTec Cross-Checks LS strip in each pack, peel-pouch or tray for sterilization. Place strip at a location in the package considered most difficult for steam to reach. For larger packs, place strips at several locations in pack.
2. Sterilize package as usual in a steam sterilizer.
3. Following sterilization cycle, verify that indicator check marks on the ends of the strip have turned from purple to green.
4. If indicator is dark purple, it suggests inadequate sterilization conditions and item(s) must be re-sterilized with new indicator strip(s).

SAFETY PRECAUTIONS

- If there is any doubt about sterility of item, it must be considered NOT sterile.
- Do not use any strips whose indicator marks are not Purple in color prior to exposure to steam sterilization processing.
- Cross-Checks LS indicator strips must NOT be used in place of a biological indicator, but can be used in conjunction with a standard infection control monitoring protocol such as that described by the AAMI Standards and Recommended practices: Good Hospital Practice: Sterilization and Sterility Assurance.

STORAGE

Steritec Cross-Checks LS steam sterilization indicator strips should be stored between 50°-100°F (10°-38°C) and at normal humidity (10-90% R.H.) in the re-sealable bag provided. No special storage conditions are necessary after exposure to steam sterilization conditions.

EXPIRY DATE

The expiry date is printed on the product packaging.

LOT NUMBER

A unique identification code, [LOT], is printed on each indicator strip and packaging labeling.

INTERFERING SUBSTANCES OR CONDITIONS

There are NO KNOWN INTERFERING SUBSTANCES OR CONDITIONS that could affect the intended use of the indicator or adversely affect the indicator performance.

RELEASE OF TOXIC SUBSTANCES

The indicator releases NO KNOWN TOXIC SUBSTANCES in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

DECLARATION OF CONFORMITY

SteriTec Products Mfg. Co., Inc. declares that the Cross-Checks LS Indicator conforms to all applicable portions ISO 11140-1 pertaining to Class 4 Multi-variable Indicators.

SteriTec Products Mfg. Co., Inc.
74 Inverness Drive East
Englewood, CO 80112 USA
(303) 660-4201 FAX: (303) 660-4213

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ID&CO S.r.l.