



Wash-Checks Cleaning Monitor

REF WC 101 & WC 101/50

Directions for Use

INDICATIONS FOR USE

The Wash-Checks Disposable Cleaning Monitor is designed for monitoring cleaning efficiency during the wash phase in hospital washer/disinfectors.

INSTRUCTIONS FOR USE

1. Insert a Wash-Checks monitor (WC 101) into a Wash-Checks Holder (WC 102) completely. The front side of the monitor with red test soil will be visible in the holder window.



2. Position Wash-Checks Holder face up inside a basket. Place basket on a shelf in the washer. Vary positions and shelves of Wash-Checks placement cycle to cycle.
3. Run a complete washing/disinfection cycle.
We recommend monitoring every shelf on every washer each day for a machine release and using a minimum of one Wash-Checks per load for routine monitoring throughout the day.
4. At the end of a washer/disinfecter cycle, remove the Wash-Checks monitor from the Holder and read the result.
Caution: Wash-Checks Holder and monitor strip may still be hot! Remove with care.

Monitor Results Interpretation:



Untreated monitor or critical / poor result.



Insufficient result.



Optimum result.

5. You may also record the location of monitor placement in the washer on the Wash-Checks with a fine tip waterproof marker.



Place a hash mark or circle representing the position of the Wash-Checks on the location rectangle that simulates an imaginary washer basket. This allows for placement tracking.

6. Staple the Wash-Checks onto the Wash-Checks Record Sheet and complete the appropriate information. Retain as a permanent record.

STORAGE

- Store Wash-Checks monitors inside the protective foil bag provided at room temperature 50° - 100°F (10° - 38°C).
- Do not remove Wash-Checks monitors from foil bag until ready for processing.
- Keep away from disinfectants and fumes.

EXPIRY DATE

The expiry date is printed on the product packaging.

SteriTec Products, Mfg. Co., Inc.
74 Inverness Drive East
Englewood, CO 80112
(303) 660-4201 FAX (303) 660-4213