

# WASH-CHECKS™ U

Disposable Ultrasonic Cleaning Monitor

Catalog No. WC 108

- **CHECK YOUR ULTRASONIC CLEANER EFFECTIVENESS WITH A CONSISTENT MONITOR ON A ROUTINE BASIS**

When cleaning conditions are effective, the test soil on monitor is dissolved.

The presence of a large amount of blue test soil remaining on the device means a serious deficiency in ultrasonic cleaning efficiency.

- **LOW COST MONITOR PROMOTES ROUTINE USE**

We recommend monitoring the ultrasonic cleaner once daily with an empty load for a machine release and periodically throughout the day with instruments for a load release.

- **USE WITH WASH-CHECKS HOLDER**

Save money by using the same holder for your Wash-Checks—Wash-Disinfectant monitors and your Wash-Checks U—Ultrasonic monitors.

Wash-checks U have a special edge that assures you insert the monitor the correct amount so that the test soil is fully exposed.

## ORDERING INFORMATION

PART #	PACKAGING	
WC 108	MONITORS	50/BAG—5BAGS/CASE
WC 102	HOLDERS	6 PER CASE

### BEFORE WASHING

**Wash-Checks U**  
Ultrasonic Monitor



### INEFFECTIVE WASHING

**Wash-Checks U**  
Ultrasonic Monitor




### AFTER EFFECTIVE WASHING

**Wash-Checks U**  
Ultrasonic Monitor



### HOLDER

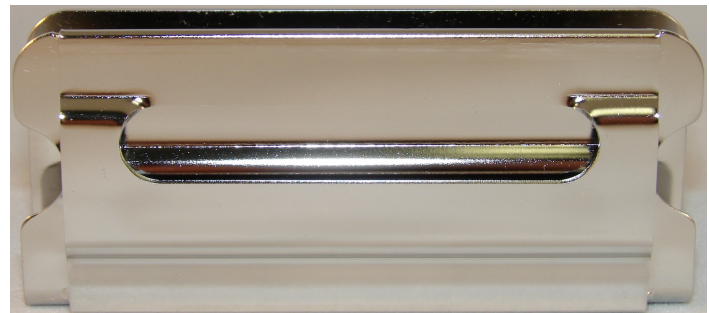

# Steri-Tec

### PRODUCTS, INC.

74 INVERNESS DRIVE EAST

ENGLEWOOD, CO 80112 • USA

(303) 660-4201 • Fax: (303) 660-4213

Website: <http://www.steritecproducts.com>

E-Mail: [info@steritecproducts.com](mailto:info@steritecproducts.com)